





Main: (587) 323-0022 Toll Free: 1 844-323-0022
sales@westechhealth.com
www.westechhealth.com

BUG!

Safe, modular & versatile.

Bug by Ormesa is a stroller with a tilt-in-space function incorporated in the seat. .

This aid is highly engineered and versatile; the wide range of adjustments makes Bug a **long-lived pushchair**, able to accompany the child during the entire growth process.

Bug is available in two sizes: Small and Medium.



BUG with base 869 passed CRASH TEST according to the **ISO 7176/19** and **ANSI RESNA WC/19**.

PRODUCT
MARKED **CE**

BUG is a **MEDICAL DEVICE** that **COMPLIES WITH THE SAFETY REQUIREMENTS** defined with the **REGULATION (EU) 2017/745**



Modular Seating System



Quick and simple to adjust for optimum posture

The seating adjustment hinges are close to the user's joints (coxo-femoral, knee & ankle joints). This structural feature is a major contributing factor to the child's well-being, as **the posture is not changed during tilting and reclining operations.**

Tilt in space and reclining

Tilting and reclining of the backrest make it possible to get the **ideal support position**. They are regulated by two independent gas springs that makes adjustment operations particularly gentle & smooth. **At fully reclined, the chair assumes a nearly supine position.**



Reversible Seat

face to face with the child

The seat of Bug size Small with 869 base is reversible. It can be used in **two directions: forward facing or facing you position** (mother direction).

The forward direction will favor the visual **exploration of the surrounding environment**, the mother direction instead the **visual contact between the child and the adult**.

forward facing



outdoor use

Adjustable on 4 bases and any flat surfaces

869 four-wheels outdoor base

Light solid wheels, sliding on all surfaces, with direction locks, independent drum brakes.

Front wheels: 17 cm.

Rear wheels: 25 cm (Small) and 30cm (Medium).



BUG with base 869 passed CRASH TEST according to the **ISO 7176/19** and **ANSI RESNA WC/19**.

Adjustable on 4 bases and any flat surfaces

856 indoor high-low feeding base

With 10 cm solid pivoting wheels and brakes. Simply and safely height adjustable, with pedal gas spring. Height goes from 34 to 61 cm.



851 indoor high-low compact base

With cm.7.5 solid pivoting wheels and brakes. Simply and safely Height adjustable, with gas spring. Height goes from 23 to 39 cm.



indoor use

850 indoor compact floor base

It allows the development of activities on the ground, maintaining maximum stability and safety.



Main: (587) 323-0022
Toll Free: 1 844-323-0022
sales@westechhealth.com
www.westechhealth.com








Headrest

incorporated in the backrest. Your child's head is always supported during the tilt adjustments of the seat or of the backrest, also without accessories.



Locking/unlocking system

on a flat surface that ensures a stable positioning well visible and extremely safe. The release lever has a safety button to avoid the accidental release of the seat from the frame.

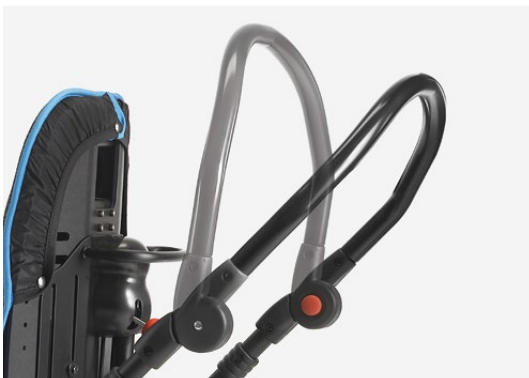

Push handle

adjustable in tilt through red buttons. It helps the care-giver in handling it even when Bug is tilting and/or reclining. (Only for 869 four-wheel outdoor base Bug).



Backrest

concave, rigid, reclinable and adjustable in height.



Tilting seat

with gas spring and adjustable in depth, width and rotation. The seat of Bug size Small with 869 base is reversible.



Leg rest

adjustable in tilt, with hinges close to the knee joints.

Tip-up foot rest

adjustable in height and tilt for the foot flex-extension.



Padded, ergonomic, removable upholstery fixed with buttons and made by “made in Italy” high quality fabric

dark grey
pink



dark grey
pink & light blue



dark grey
green



dark grey
green & light blue



light grey
pink



Padded, ergonomic, removable upholstery fixed with buttons and made by “made in Italy” high quality fabric

light grey
pink & light blue



light grey
green



light grey
green & light blue



- > **Fire-proof:** its fire-proof properties are inherent to the yarn and they do not come from any additional treatments.
- > **Recyclable:** production cycle with low environmental impact.
- > **Martindale method:** maximum resistance to rubbing: 100.000 cycles. UNI ENI ISO 12947.
- > **Oeko-Tex Certificate:** for all fabrics that are in direct contact with the skin, to guarantee quality and safety for health.
- > **Long-lasting and highly resistant colours:** they are resistant to sweat, light, dry cleaning and washing machine (see the Use and Instruction manual).



CERTIFICATE
MED annex
A.1/3.19-20-21



Main: (587) 323-0022 Toll Free: 1 844-323-0022
sales@westechhealth.com | www.westechhealth.com

Components



863 Headrest

with parietal supports. Adjustable in height, width, tilt and forward-backward.



852 Headrest

with parietal supports. Adjustable in height and width.



942 Ergonomic headrest

Adjustable in height, tilt and forward-backward with adjustable sides. Size L (large) available for special case.



835 Shaped headrest



934 Suspension headrest attachment

Available only for 863 headrest.



838 Lateral supports

adjustable in height and width.

Components



868 Wrappable and flexible trunk supports
Adjustable in height and width.




834 Padded abduction block
Adjustable in depth.



834R Padded abduction block
Adjustable in depth and abduction.



853 Vest harness

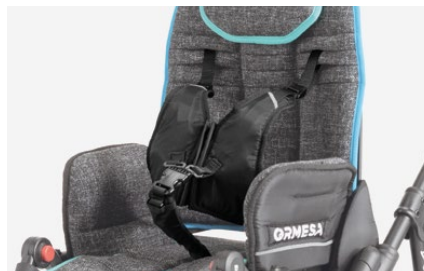

903 Five point vest harness



906 Five point harness

Components




894 45° pelvic belt




920 Four Point belt



827 Foot straps

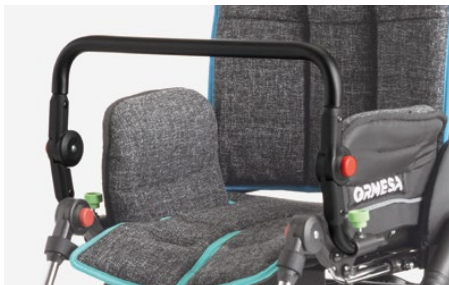

839 Front handle
Adjustable in tilt.

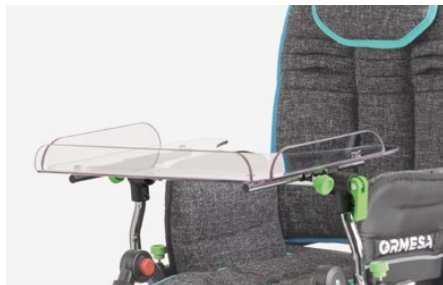

824 Transparent tray
wrappable, with recess. Adjustable in tilt.



824B Wide tray for indoor base
adjustable in tilt and depth.

It is made of resistant and hygiene material, no edge design, it does not retain any residue of food.

Components



905 Hand brake
for 869 base.



935 Oxygen bottle holder for 869 base

Maximum dimensions of the oxygen bottle: height 600 mm (valve included) and diameter 140 mm.
The frame can be folded without removing the accessory.



911 Vent tray
for carrying medical equipment. For 869 base. **The frame can be folded without removing the accessory.**



819 Canopy
in combination with the upholstery. Resistant and easily adjustable.



825 Rain cover
(to fix at the canopy 819) for 869 base.



818 Thermal cover

Components



892 Covering for footrest
padded and removable.



858 Net basket for 869 base



921 Sibling rear footboard
Placed behind the stroller. For children of at least 15 months. Max. load 20 kg (44 lbs.) For 869 base.



922 Bottle holder
For 869 base.




891 Set of 4 tie down hooks
for the forward-facing transport of the pushchair in a motor vehicle (private/cars, buses..). Only for 869 base.
See User and Maintenance Manual provided with the aid.



Bug | 869 base sizes and weight

Modular postural system on four-wheels outdoor base.

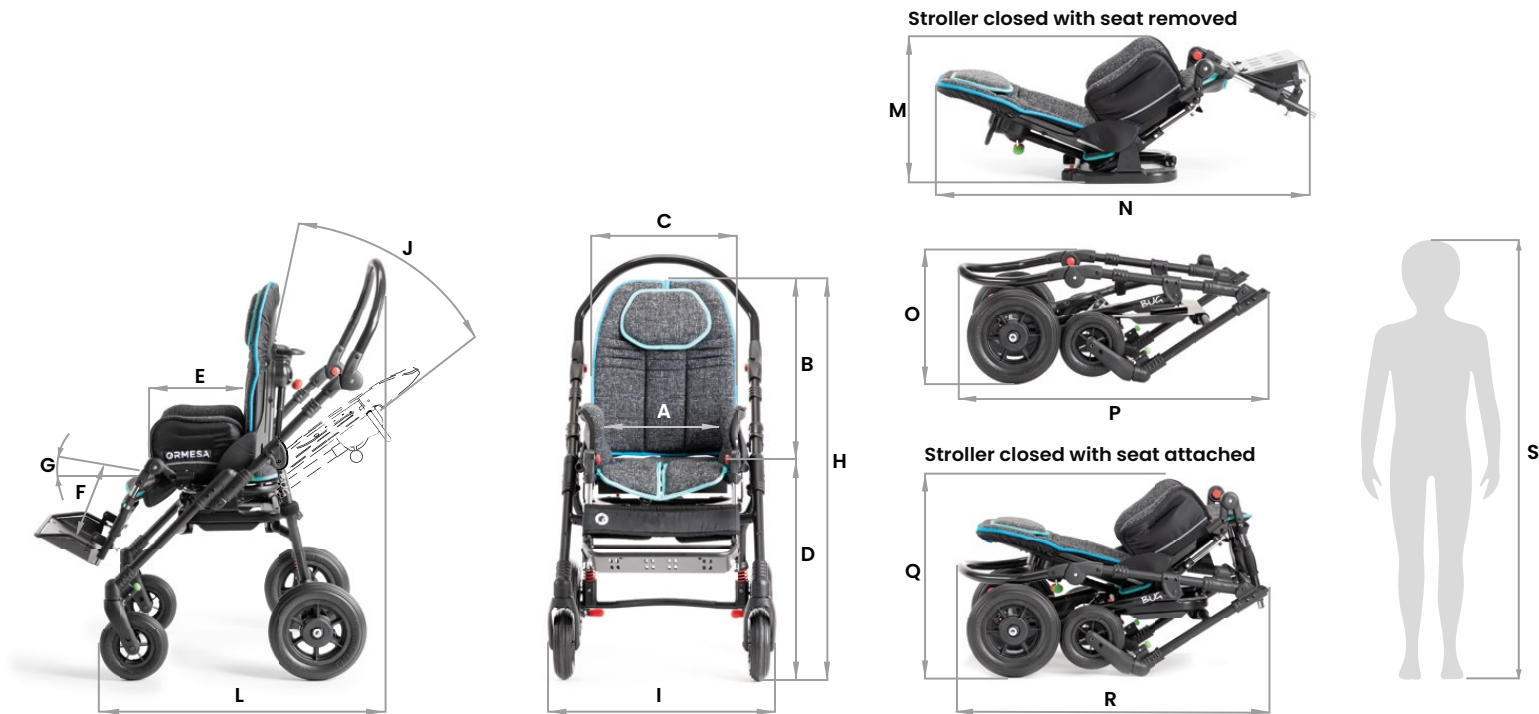

	A	B	C	D	E	F	G	J	H	I	L	M	N	O	P	Q	R	S	max. load	frame weight	seat weight	total weight
Small	20-30 cm	52-62 cm	37 cm	52 cm	20-30 cm	14-30 cm	0°-35°	35°	100-110 cm	57 cm	67-96 cm	35 cm	88 cm	33 cm	82 cm	50 cm	81 cm	90-120 cm	35 kg	9,5 kg	8,5 kg	18 kg
	7,8-11,8 in	20,4-24,4 in	14,5 in	20,4 in	7,8-11,8 in	5,5-11,8 in			39,3-43,3 in	22,4 in	26,3-37,7 in	13,7 in	34,6 in	12,9 in	32,2 in	19,6 in	31,8 in	35,4-47,2 in	77,1 lb	20,9 lb	18,7 lb	39,6 in
Medium	25-36 cm	60-75 cm	46 cm	55 cm	30-40 cm	20-38 cm	5°-30°	40°	112-127 cm	62 cm	80-107 cm	37 cm	112 cm	39 cm	94 cm	52 cm	94 cm	115-140 cm	45 kg	11 kg	12,7 kg	23,7 kg
	9,8-14,1 in	23,6-29,5 in	18,1 in	21,6 in	11,8-15,7 in	7,8-14,9 in			44-50 in	24,4 in	31,4-42,1 in	14,5 in	44 in	15,3 in	37 in	20,4 in	37 in	45,2-55,1 in	99,2 lb	24,2 lb	27,9 lb	52,2 lb

Hyphen-separated numbers indicate the minimum and maximum range of adjustment

Bug | 856 base sizes and weight

Modular postural system on indoor high-low feeding base.




	A	B	C	D	E	F	G	J	H	I	L	M	N	O	P	max. load	frame weight	seat weight	total weight
Small	20-30 cm	52-62 cm	37 cm	47-75 cm	20-30 cm	14-30 cm	0°-35°	35°	95-132 cm	56 cm	34-61 cm	80 cm	40 cm	25 cm	90-120 cm	35 kg	14,5 kg	8,5 kg	23 kg
	7,8-11,8 in	20,4-24,4 in	14,5 in	18,5-29,5 in	7,8-11,8 in	5,5-11,8 in			37,4-51,9 in	22 in	13,3-24 in	31,4 in	15,7 in	9,8 in	35,4-47,2 in	77,1 lb	31,9 lb	18,7 lb	50,7 lb
Medium	25-36 cm	60-75 cm	46 cm	47-75 cm	30-40 cm	20-38 cm	0°-25°	40°	106-148 cm	62 cm	34-61 cm	83 cm	45 cm	32 cm	115-140 cm	45 kg	16,2 kg	12,7 kg	28,9 kg
	9,8-14,1 in	23,6-29,5 in	18,1 in	18,5-29,5 in	11,8-15,7 in	7,8-14,9 in			41,7-58,2 in	24,4 in	13,3-24 in	32,6 in	17,7 in	12,5 in	45,2-55,1 in	99,2 lb	35,7 lb	27,9 lb	63,7 lb

Hyphen-separated numbers indicate the minimum and maximum range of adjustment

Bug | 851 base sizes and weight

Modular postural system on indoor high-low compact base.

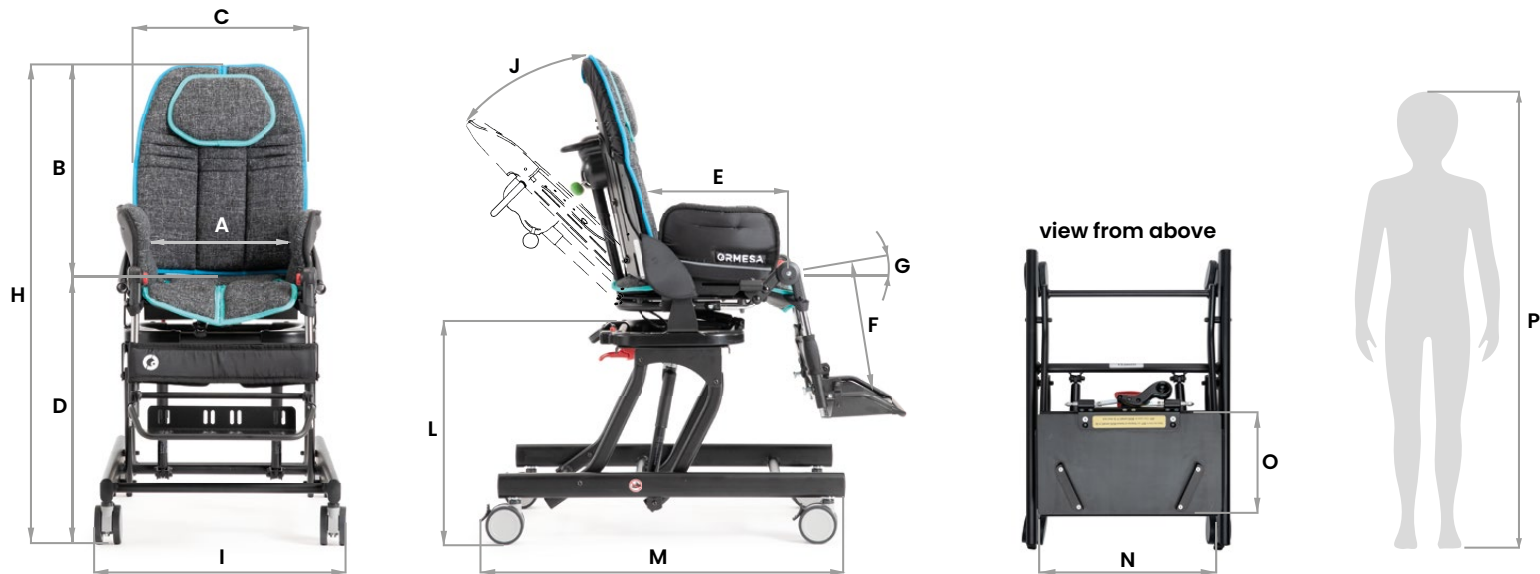

	A	B	C	D	E	F	G	J	H	I	L	M	N	O	P	max. load	frame weight	seat weight	total weight
Small	20-30 cm	52-62 cm	37 cm	34-50 cm	20-30 cm	14-30 cm	0°-35°	35°	84-110 cm	50 cm	23-39 cm	72 cm	41 cm	23 cm	90-120 cm	35 kg	8,8 kg	8,5 kg	17,3 kg
	7,8-11,8 in	20,4-24,4 in	14,5 in	13,3-19,6 in	7,8-11,8 in	5,5-11,8 in			33-43,3 in	19,6 in	9-15,3 in	28,3 in	16,1 in	9 in	35,4-47,2 in	77,1 lb	19,4 lb	18,7 lb	38,1 lb
Medium	25-36 cm	60-75 cm	46 cm	34-50 cm	30-40 cm	20-38 cm	0°-25°	40°	95-126 cm	55 cm	23-39 cm	76 cm	46 cm	28 cm	115-140 cm	45 kg	10 kg	12,7 kg	22,7 kg
	9,8-14,1 in	23,6-29,5 in	18,1 in	13,3-19,6 in	11,8-15,7 in	7,8-14,9 in			37,4-49,6 in	21,6 in	9-15,3 in	29,9 in	18,1 in	11 in	45,2-55,1 in	99,2 lb	22 lb	27,9 lb	50 lb

Hyphen-separated numbers indicate the minimum and maximum range of adjustment

Bug | 850 base sizes and weight

Modular postural system on indoor compact floor base.



	A	B	C	D	E	F	G	J	H	I	L	M	N	O	P	max. load	frame weight	seat weight	total weight
Small	20-30 cm	52-62 cm	37 cm	17 cm	20-30 cm	14-30 cm	0°-35°	35°	68-78 cm	46 cm	6 cm	61 cm	41 cm	23 cm	90-120 cm	35 kg	4,5 kg	8,5 kg	13 kg
	7,8-11,8 in	20,4-24,4 in	14,5 in	6,6 in	7,8-11,8 in	5,5-11,8 in			26,7-30,7 in	18,1 in	2,3 in	24 in	16,1 in	9 in	35,4-47,2 in	77,1 lb	9,9 lb	18,7 lb	28,6 lb
Medium	25-36 cm	60-75 cm	46 cm	17 cm	30-40 cm	20-38 cm	0°-25°	40°	77-92 cm	51 cm	6 cm	63 cm	46 cm	28 cm	115-140 cm	45 kg	5,5 kg	12,7 kg	18,2 kg
	9,8-14,1 in	23,6-29,5 in	18,1 in	6,6 in	11,8-15,7 in	7,8-14,9 in			30,3-36,2 in	20 in	2,3 in	24,8 in	18,1 in	11 in	45,2-55,1 in	99,2 lb	12,1 lb	27,9 lb	40,1 lb

Hyphen-separated numbers indicate the minimum and maximum range of adjustment



Integrity | Trust | Honesty | Knowledge | Respect



Westech Health Care Ltd.

154 - 5255 McCall Way NE, Calgary, Alberta T2E 7J5
Main: (587) 323-0022 Toll Free: 1 844-323-0022
sales@westechhealth.com
www.westechhealth.com



"Our goal is to help Canadians of all ages live independently with dignity and respect by offering innovative health care products."